PJAMA BEDWETTING ALARM

User Instruction Manual

Instructions available in
14 languages.

GB DryGuardians-APP

We recommend using the DryGuardians App. Enhance your bedwetting treatment with the DryGuardians App, developed with Swedish healthcare experts to support your treatment of bedwetting.

- **Free Download:** Easy to get started.
- **Quick Setup:** Fast registration process.
- **Track Results:** Log nightly outcomes.
- **Personalized Feedback:** Get tailored insights.
- **Expert Tips:** Access helpful advice.

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FCC WARNING

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for Class B digital devices, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and the receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

PJAMA BEDWETTING ALARM

INTRODUCTION

The Pjama bedwetting alarm is used to treat bedwetting (enuresis nocturna) in children from the age of 5 years. The treatment can usually be carried out in the child's home. The Alarm Sensor connects to the Pjama Treatment Garments that has built-in sensor threads sewn into it.

CAUTION

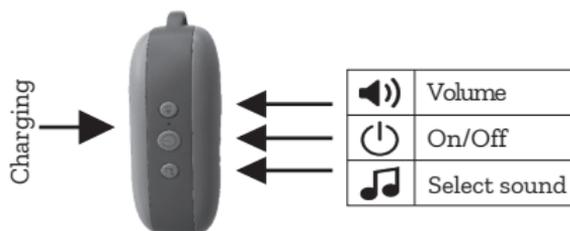
- Product should be used by individuals aged five and older, or as recommended by a physician.
- Pjama Bedwetting Alarm is intended for use as an enuresis alarm. Make sure that no parts of the Pjama Bedwetting Alarm are put in the mouth for any reason. This could cause choking or other harm.
- The Pjama Bedwetting Alarm is a medical device that should be treated with care.
- Do not hold an alarming speaker within 4 inches (10 cm) of the ear. This could cause damage to the ear.
- Do not charge the speaker unit during the night or when not at home.
- To extend battery life, disconnect the sensor from pants when not being used.

DEVICE DESCRIPTION

CONTENTS

Sensor	Sensor that is connected to the Pjama underwear via two buttons.
Audio Device	The unit is wirelessly connected to the sensor. In the event of a urine leak this device will make a sound to wake up the child.
Charging Cable	Audio device charging cable. Connection between audio unit and charger.
Manual	Instructions on how the alarm and its components work.
Treatment Garment	Underwear with sewn-in sensor threads. The threads are connected to the sensor via 2 buttons on the garment. Pjama Treatment Pants & Shorts can replace the treatment underwear or be used as a complement textile garment. If the Alarm is bought as a separate unit, textile products are not included.

DIAGRAM

INSTRUCTIONS FOR USE

FOR BEST RESULTS

The child should be motivated and want to get dry. A parent needs to help the child to wake up, especially at the beginning of the treatment.

CHARGING OF AUDIO DEVICE

NOTE: Charge the Audio Device before use. Use the supplied USB cable. The sensor does not need to be charged.

Charging via:	Time- fully charged:
Computer	4 - 6 hours
AC Adapter 5V 1.0A	3 hours

During charging a red LED light flashes. When Charging is complete a red LED light will be lit constantly.

HOW TO OPERATE

1. Turn on the alarm by holding down the  button on the Audio Device. A blue LED light starts flashing to the left of the  button and you will hear a sound that the alarm is activated. The red LED flashes when the unit needs charging.
2. Press the  button and select one of the seven different alarm sounds. The last played sound will be the sound alarming. Adjust the volume with the  button.
3. Test the alarm unit. Short-circuit the Sensor contacts by using a metallic object. The range between the Sensor and the Audio Device is approximately 10m. If your alarm does not seem to function correctly, see the instructions under '**Pairing of Sensor**' below.

Short-circuit the contacts on the sensor using a metal object such as scissors.

4. Attach the sensor to the buttons located in the pocket on the inside of the Pjama Treatment Pants or on the outside of the underwear. Make sure it is securely fastened. The alarm is now ready to use.
5. Press the  button to turn off the alarm. Hold down the button to completely turn off the alarm.
6. Disconnect the sensor from the Treatment Pants or Underwear **immediately** after the alarm has sounded. It prevents the alarm from sounding again and saves the battery time on the sensor. Wipe off the sensor if it gets wet. **DO NOT** wash the sensor in a washing machine.

CLEANING

The Sensor rarely gets wet in the event of urine leakage as it is attached

high up on the textile garment. If necessary, the Sensor and Audio device can be cleaned with soap and water or disinfected with an alcohol-based disinfectant. The Pjama garments can be machine washed at 60 °C. Carefully read additional washing instructions on the label on the inside of the garment.

PAIRING OF SENSOR

If your alarm device does not appear to be functioning correctly, please try to pair the Sensor with the Audio Device by doing the following:

Pair the sensor with the Audio Device by pressing the  button and the  button on the Audio Device simultaneously for 3 seconds. Then short circuit the buttons on the Sensor with a metal object. The Audio Device indicates that the Sensor is paired with it by playing the alarm signal.

Download the DryGuardians Application from Google Play or Apple App Store on your phone or tablet and follow the instructions in the application to connect the sensor to the device.

STORAGE AND HANDLING

To maintain the alarm's effectiveness, follow these storage guidelines:

Room Temperature: Store the device in a stable, room-temperature environment, avoiding extremes of heat or cold.

Sensor Care: Detach the Sensor from the garment immediately after activation. Avoid storing it on metal surfaces to prevent interference or damage.

Environmental Precautions: Keep the device dry and away from direct sunlight, dust, and chemicals.

WARRANTY

The Pjama bedwetting alarm has a one-year warranty from the buyer's purchase date of the alarm. You must be the original buyer and show evidence of your purchase. This warranty does not apply if malfunction is caused by an accident or similar incident or injury, water damage, negligence, abnormal use or failure to comply with the instructions in this manual.

REGULATORY INFORMATION

The Pjama Bedwetting Alarm, model 9031, meets all relevant European regulations, classified as a Class I medical device under EU MDR 2017/745. The CE mark confirms compliance with essential EU requirements.

REF 9031 MD        ERIC

For more information see: pjama.co.uk/product-manuals

36 Old School House Britannia Road,
Kingswood, Bristol, BS15 8DB, UK

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Pjama AB