Pjama AB Lagomvägen 36 SE- 218 53 Klagenham, Sweden

Manual no: 1916

Version: 4.0

Date: 2025-02-27

FCC WARNING

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for Class B digital devices, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determind by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and the receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

GB
INTRODUCTION
Pjama Connect is used to treat bedwetting (enuresis nocturna) in children from the age of 5 years. The treatment can usually be carried out in the child's home. The Sensor connects to the Pjama Audio device or DryGuardians App. The sensor is to be fixated in a party liner.

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GB PJAMA CONNECT BEDWETTING ALARM

INSTRUCTION

For best results: The child should be motivated and want to get dry. A parent needs to help the child wake up, especially at the beginning of treatment.
NOTE: The sensor does not need to be charged. Its equipped with Bluetooth® and the sensor communicates directly with the **Pjama Audio Device** or the **DryGuardians App**.

- Start by downloading the DryGuardians Application from Google Play or App Store.
- Register in the application.
- Test the Sensor with the App or the Audio Device. Short circuit the sensor by placing a metallic object or a drop of water on the metal surface. This will trigger the alarm. The sensor range is 10 meters (30 feet).
- The sensor is now ready for use. **Use a panty liner, make an incision and insert the sensor.** Place in the underwear.
- Wipe the sensor after use.

The Sensor can be cleaned with soap and water or disinfected with alcohol-based disinfectants.

WARRANTY

The Pjama Connect has a one- year warranty from the buyer’s purchase date of the alarm. You must be the original buyer and show evidence of your purchase. This warranty does not apply if malfunction is caused by an accident or similar incident or injury, water damage, negligence, abnormal use or failure to comply with the instructions in this manual.

- ⚠
CAUTION
 - Never place the sensor directly against the skin.
 - Pjama Connect is intended for use as an enuresis alarm.
 - Make sure that no part of the Sensor is put in the mouth for any reason. This could cause choking or other harm.
 - The Pjama Connect Bedwetting Alarm is a medical device that should be treated with care.
 - Do not hold an alarming Audio Device within 10 cm (4 inches) of the ear. This could cause damage to the ear.
 - To extend battery life and prevent corrosion of the sensor surface, remove the sensor from the wet environment immediately after an accident and when not in use.

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